



TImask Suisse Sagl

INSTRUCTION MANUAL

Filtering half mask:

- **Personal protection equipment type FFP2 NR**
- **Certification number EPT 0477.PPE. 21/4489**

Product name TIME



Product: Single use NR (non-reusable)


Packaging: 5 units per package

Box: Contains 4 packages for a total of 20 units

NON-STERILE yet in suitable sanitary hygienic condition

INTENDED USE: for the general public and in work environments

Ligornetto this 14 day of June 2021

Legal representative,  Moreno Lazzaroni

WRITTEN BY TImask Suisse Sagl

Via Michele Roggia 2, 6853 Ligornetto, Switzerland CHE-229.952.023 IVA

Tel. +41 91 647 1717 / email: commerciale@timask.ch



INSTRUCTION MANUAL

Regulatory References:

- Regulation EU 2016/425 of March 9, 2016 on personal protective equipment
- EN 149:2001 + A1:2009

Fields of Application

In accordance with the EN 149:2001+A1:2009, the filtering half mask TIME FFP2 NR is intended to provide protection of the respiratory tract against solid and liquid aerosol and droplets. This filtering half mask provides 99,04% filtering performance.

How to correctly wear the filtering half mask

- This filtering half mask does not protect against life threatening levels of contaminants and should be worn only in accordance with health and safety regulations.
- Ensure the filtering half mask is intact and is not expired (the expiration date is shown under the box) and verify the filtering half mask is adequate for its intended use according to the product used and its concentration.
- Wash your hands.
- Open packaging immediately before use.
- Identify the front from the back.
- Hold the filtering half mask in your palm, with the nasal bridge facing up and the elastic bands hanging down.
- Position the filtering half mask under your chin and cover your mouth and nose.
- Hook the elastic bands around your ears creating a tight fit on your face (if the elastic bands are too long, shorten them by making a knot in each one).
- Using both hands, press the nasal bar around the nose to secure a tight fit.
- Check the hermetic seal by breathing heavily and check that there is no air leakage around the perimeter of the filtering half mask.
- Readjust if necessary.
- Try to avoid touching the filtering half mask while wearing it; if you must touch it, wash your hands before and after.

How to correctly remove the filtering half mask

- Wash your hands.
- Remove elastic bands from behind your ears.
- Pull the filtering half mask away from your face with both hands.
- Dispose the used filtering half mask in a garbage bin making sure not to touch the interior of the filtering half mask.
- Wash your hands.

Restrictions of use

- This filtering half mask must not be used for more than one day (NR non-reusable), the filtering half mask should not be worn for more than one workshift.
- This filtering half mask can be used only once.
- Do not use this filtering half mask for any other purpose than that defined in this leaflet.



WARNINGS

- Before wearing this filtering half mask, you must have previously received adequate training and have read this information leaflet.
- Be sure to wear the filtering half mask in well ventilated areas with the presence of oxygen greater than 19.5%. Never wear the filtering half mask in explosive atmospheres.
- The filtering half mask must be perfectly positioned to guarantee the expected level of protection.
- Do not use if facial hair, beard or sideburns prevent the perfect fit or allow the possibility of leaks.
- Leave the risk area if breathing becomes difficult and/or if you feel dizzy or unwell.
- Do not modify this filtering half mask in any way and throw it away if it is damaged.
- If, while wearing the filtering half mask, you have difficulty breathing, shortness of breath due to external causes, for example air saturated with dust, replace the mask immediately.
- **The filtering half mask is flammable – keep away from fire and flames.**
- **The filtering half mask should be stored in a dry and well-ventilated area.**

Side Effects

- Although there are no known cases of an allergic reaction to the skin from wearing this filtering half mask, should this occur, leave the risk zone, remove the filtering half mask and consult a physician.
- If you experience redness, swelling and/or itching after, discontinue use and consult a physician.
- This filtering half mask complies with the regulations on PPE and not those on medical devices.
- If this filtering half mask is not compliant, for example it has broken elastics, protruding nose bars, etc., this PPE must be discarded and replaced with a compliant one.

Instructions for storage and transport

When not in use, transport in the original packaging.

This filtering half mask should be stored in its original packaging, in a dry, cool and well-ventilated area, protected from frost and light (between -10C - +50C, 14F – 122F, R.H. <80%)

Methods of Disposal

Discard the filtering half mask utilizing the separate waste collection container at your disposal.



Additional information

The filtering half mask is subject to an assessment procedure under the supervision of **Eurofins Product Testing Italy Srl. (EPT), via Cuornè 21, 10156 Turin, Italy, CE 0477.**

Certification number EPT 0477.PPE.21/4489

The declaration of conformity is available at the following web address:

www.timask.ch

TIme

FFP2 NR CE 0477
EN 149:2001+A1:2009

TImask Suisse

WRITTEN BY TImask Suisse Sàgl,
Via Michele Roggia 2, 6853 Ligornetto, Switzerland
CHE-229.952.023 IVA
Telephone: +41 91 647 1717 Email: commerciale@timask.ch